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Operating Instructions  
Model saw G2 Concept

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**We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new G2 Concept all purpose model saw.**

**We wrote these operating instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.**

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## **Index**

1. List of contents - accessories
2. Range of applications
3. General information  
**Caution !**
4. Picture G2 Concept - operating elements
5. Setting up
6. Operating the unit
7. Exchanging the saw blade
8. Calibrating laser beam
9. Maintenance and care
10. Possible faults
11. Technical data
12. Declaration of conformity

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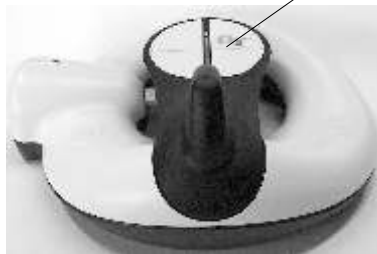
3105 (3106)

1932

## 1. List of contents - accessories/spare parts

**G2 Concept**

**Art.No.: 1500**

Model saw complete with line marking laser (1)

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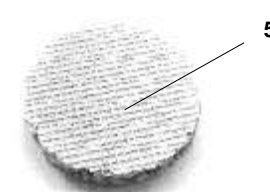
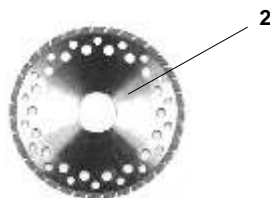
Laserklasse: 3A

## 2. Range of Applications

Until now, many models have necessitated saw cutting by hand, due to the limited space between prepared teeth or irregularities in the alignment of the plaster dies.

With the new G2 Concept, these sawing operations can be carried out mechanically with highest precision, because the G2 Concept is operating from below.

The G2 Concept provides for the optimum saw cutting of all models without damaging the preparation lines. Soft starting, instant stopping and two-hand sensor technology are guarantee for your.

Conditions of environment: - interior 10° - 40°C

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### 3. General Information

- 3.1 Ascertain that your mains supply coincides with the data in the rating plate.
- 3.2 The model saw G2 Concept is **not** suitable for the following applications:
  - in areas where there is a risk of explosion
  - on patients
  - for working on moist materials
- 3.3 Insure that all regulatory requirements are observed during use
  - always wear protective glasses
  - take care for good illumination
  - use dust suction

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WEEE-Reg.-Nr. DE 78620387

### Caution

- Use only with original SCHICK saw blade
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- SCHICK do not guarantee the model saw G2 Concept should it not have been used in accordance with the operation instructions.

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

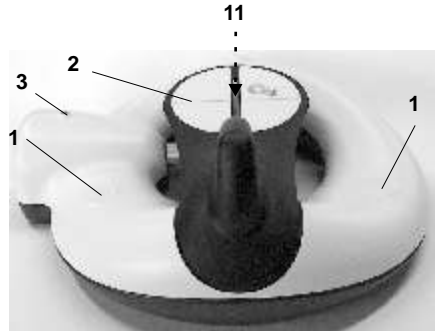
#### 4. Picture G2 Concept - operating elements

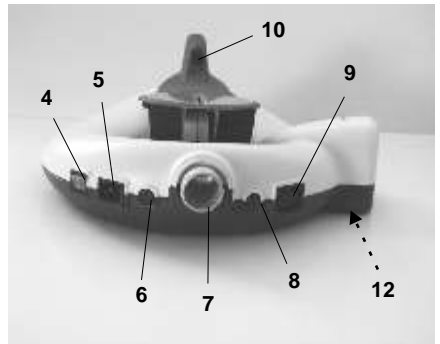- 1 - hand rest
- 2 - saw table
- 3 - control lamp
- 4 - mains switch
- 5 - socket for mains cable
- 6 - fuse 800 mA T/230V (2A T/100V/115V)
- 7 - extractor socket for external dust suction
- 8 - fuse motor 8 A T
- 9 - socket to plug in external dust suction
- 10 - line-marking laser
- 11 - saw blade

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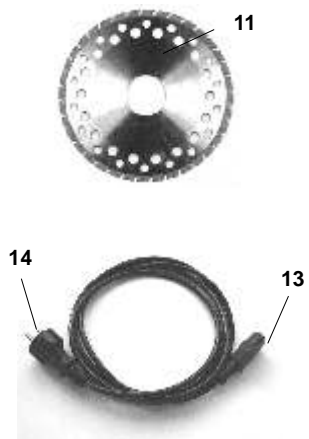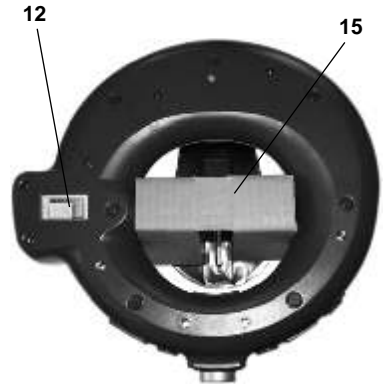
## 5. Setting up

Unpack the G2-Concept and check if complete

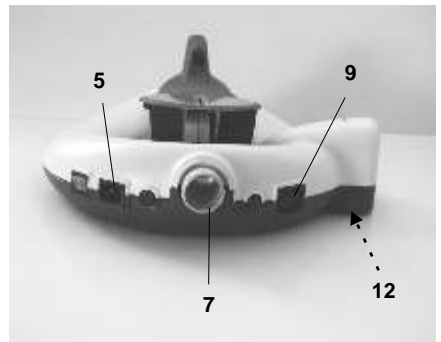
Take off carefully transportation lock (15) for model table and saw-blade

Check whether power data with the information on the identification plate (12) corresponds

Mains cable (13) in the mains socket (5) and afterwards power plug (14) in mains socket or in the external suction socket.

Plug in the tube of the external suction (7)

In use of an external suction without controller plug mains cable of the external suction with

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## 6. Operating the unit

- Turn on device at the main switch in the back of the G2 Concept.
- To be sawed press the model with the Pins downwards into the supporting plate (according to the length of your pin system use 1 or 2 supporting plates)
- Put model with supporting plates on the saw table and position with both hands.
- Pay attention that both wrists lie on the sensors and take thumb under the saw table and fix the model with forefinger and middle finger on the model table.
- Align position in the laser line accurately.
- Press model easily downwards. The saw and the external suction will be turned on automatically in operation by the inserted sensor technology.
- Press model under light pressure farther downwards. Now the saw blade dives from below by the supporting plate and into the plaster arch and produces a clean saw cut in parallel to the pins.
- If the optimal cut depth is reached, drive back the saw table slowly again in the starting position.
- Then the saw switches off itself again automatically. To work nearly dust free the external suction will run a few time longer and stop then automatically.

## 2. Security advice

Operate saw never if hand or finger are in the area of the laser line!

Risk of injury!

Saw pursue only with protection eyeglasses!

Saw pursue absolutely with corresponding dust suction!

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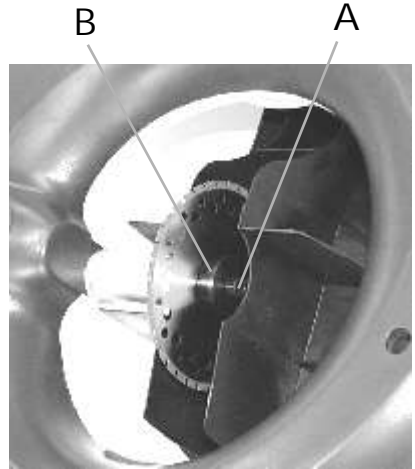
## 7. Change of the saw blade

Pull net plugs  
Decrease suction tubes  
Turn G2 Concept and place preferably on the high thighs  
Unscrew inside hexagonal screw (A) and remove fixation ring (use counter pressure with the provided tool in the intended whole (B) on the shaft)  
Exchange saw blade (Use only original SCHICK diamond discs!)  
Insert fixation disc again and scw in the inside hexagonal screw under counterholds with the intended tool

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8. Calibrating the laser beam of modelsaw  
G2-Concept (as of series number 150.416)

1. Switch the unit on.  
**Attention:** Avoid direct eye contact with the laser beam!
2. Use allen key 3mm (pic.1) horizontally and undo clamp bolt.
3. Use allen key 2,5mm with an ball-head sloping (pic.2) into levelling screw placed below the clamp bolt.
4. When screwing in the levelling screw (pic.2) the laser beam moves to the right; when screwing out the levelling screw the laser beam moves to the left.
5. Adjust laser beam in the desired position and secure clamp bolt (pic.1) by using allen key 3mm.

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Attention:  
Do not look directly  
into the laser beam  
source

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## 11. Technical Data

Dimensions:	Width: 375 m Height: 160 mm Depth: 390 mm
Weight:	4,5 kg
Power supply:	230 / 115 / 100 V
Nominal frequency:	50 / 60 Hz
Power input:	130 W
Speed range:	7.500 rpm
Diamond saw blade:	Ø 85 x 20 mm segmented
Width of cut:	0,3mm
depth of cut:	26,5mm
Laser class:	3A
Safety class:	2

This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression.

These operating instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using original spare parts and we are extending another **6 month guarantee** on repairs in our house

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Nous, GEORG SCHICK DENTAL GmbH
Georg Schick Dental GmbH